



# DAKEWE

## DP360 Stainer CS600 Coverslipper Workstation

Flawless Coverage. Flawless Results. Every Time.

## Specifications DP360 slide stainer

|                            |                                 |
|----------------------------|---------------------------------|
| Dimensions, (W × D × H)    | 1066 × 700 × 630 mm             |
| Net weight                 | Approx. 130 kg                  |
| Power consumption          | 1200 VA                         |
| Supply voltage/frequency   | 100 - 240 VAC, 50/60 Hz         |
| Total stations             | 26                              |
| Oven                       | 1; Max temperature 75 °C        |
| Reagent heating stations   | 4; Max temperature 40 °C        |
| Washing stations           | 4                               |
| Loading/unloading stations | 4 in total, flexibly adjustable |
| Reagent station volume     | 480 ± 20 ml                     |
| Protocol storage capacity  | ≥ 100 protocols with ≥ 40 steps |
| Slide rack capacity        | 30 slides                       |

## Specifications CS600 coverslipper

|                            |                                                                          |
|----------------------------|--------------------------------------------------------------------------|
| Dimensions, (W × D × H)    | 700 × 650 × 665 mm                                                       |
| Net weight                 | Approx. 89 kg                                                            |
| Power consumption          | 250 VA                                                                   |
| Supply voltage/frequency   | 100 - 240 VAC, 50/60 Hz                                                  |
| Slide dimension            | (24.8 - 26.2) ×<br>(74.8 - 76.2) mm                                      |
| Coverslip length           | 50, 55, 60 mm                                                            |
| Coverslip storage capacity | Up to 500 coverslips in 5 magazines                                      |
| Unloading station storage  | Up to 300 slides                                                         |
| Oven temperature           | 50 °C                                                                    |
| Workstation                | Can be used stand-alone or integrated with DP360 without extra connector |

# DAKEWE

Dakewe medical  
Shenzhen, China  
[www.dakewemedical.com](http://www.dakewemedical.com)  
Ver. 1.0, Rev. Jan 2025





### Enjoy walkaway process with massive capacity

500-coverslip large magazine continuously loadable and unloading station holding up to 300 slides free technicians from repetitively loading and unloading and allow for an uninterrupted workflow.



### Fast drying

Heated and well ventilated uploading station accelerates slide delivery.



### Adaptability

Seamlessly handles 50, 55, and 60 mm coverslips with adaptable magazines while offering intuitive mounting medium control for all applications, compatible with most commercially available mounting media.

## CS600 GLASS COVERSLIPPER



### Care for the users from the air they breathe


DP360 stainer activates its exhaust system to effectively purge fumes when concentrations reach critical thresholds, ensuring optimal user safety and performance.



### Hassel-free with all statuses in control

Precision level sensors continuously monitor mounting medium and cleaning reagent volumes, preventing process interruptions and eliminating the urgency of last-minute reagent replenishment. Automatic nozzle soaking and regular piping cleaning reduce the risk of blockage and bubbles, and further offers high levels of stability.

The  
integrated,  
automatic,  
pragmatic

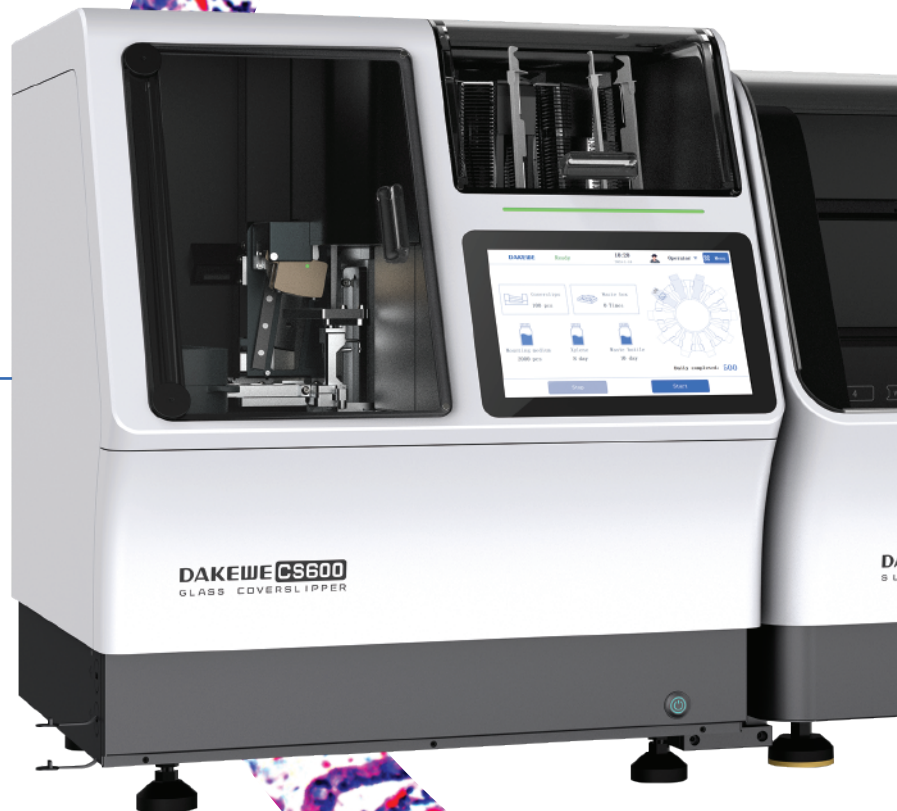

# staining workstation.

## Slide traceability



Equipped with an integrated barcode scanner, CS600 connects to Dakewe stainers to deliver complete process monitoring - tracking protocols, reagents, and operations while enabling seamless LIS integration.

## Versatility



Effortlessly handles multiple protocols simultaneously, choosing from 100 staining protocols, including histology, cytology, frozen section and more.

## Optimal staining quality



Four heating reagent stations facilitate effective staining. Specially engineered slide racks and robotic arm motions reduces cross-contamination.


## DP360 SLIDE STAINER

## Intelligently optimized homogeneity




The exclusive Automatic Adjustment Technology (AAT) on DP360 stainer with Dakewe staining reagents guarantee a consistent staining result up to 2000 slides.

1st slide

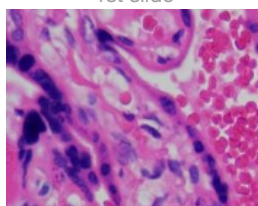

1000th slide

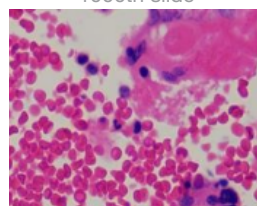

2000th slide

